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Read the user information carefully and keep it near the workstation. Employees should be taught how to use the emergency cleaning kit at least once a year. Ignoring the user information can impair the performance of the SpillKit components and be harmful to the health of the user and the environment.

Contents (Art.-No: 4004): 1 pair blue protective gloves for use with cytostatic drugs (CE 2797; EN 374), 1 pair green protective overgloves (CE 0120; EN 374, EN 388), 1 protection gown for use with cytostatic drugs (CE 2797; EN 14126), 1 FFP-3 respirator mask (CE 0426; EN 149), 1 pc safety goggles (CE 0196; EN 166), 1 pair overshoes, 1 ChemoSorb Pad, 6 pcs ISYSOFT cloths, 2 waste bags, 2 cable ties, 1 user information, 1 casualty report (German only).

1 Preparation:

Important: When cleaning a safety cabinet, leave the fan switched on!Open the carton at the perforation. Use packaging as “sweeper and shovel”. Take the parts out of the bag and lay them down in front of you for better selection. Please heed the following steps.

1. Put on the respiratory mask. Hold the mask in your hand with the straps hanging down. Guide the first strap over your head as far as the nape of your neck . Place the mask under your chin. Guide the the second strap over the back of your head. Adjust the nosepiece to fit your nose. Cover the half-mask with both hands and breathe out strongly. The mask material will move up and down. If air escapes at the edge of the mask, move the mask into the correct position and adjust the nosepiece to fit your nose. Please note that facial hair and scars can impair the fit of the mask.
2. Don the safety goggles. Use the head strap to adjust them and ensure a secure fit. Safety goggles can be worn over corrective glasses.
3. Put on the protective gown for use with cytostatic drugs.
4. Don the blue or buff-coloured protective gloves for use with cytostatic drugs and pull the cuffs over the ends of the gown’s sleeves.
5. Then don the green-coloured protective gloves over the blue or buff coloured gloves.
6. Lastly, put on the protective overshoes.

2 Cleaning / How to work with the SpillKit:

Take the “ChemoSorb Pad” and place it over the spilled liquid. The special coating of pads will bind liquids, within a short time, to a sort of jelly. Use packaging as “sweeper and shovel” to remove the cytotoxic waste. Put the complete waste into the grey-blue waste bag. Clean thoroughly with the ISYSOFT cloths and water. Put used cloth into the grey-blue waste bag.

3 Removal and disposing of protective clothing:

Take off the green overgloves by pulling the cuff over the fingers (Only the green overgloves). Put the gloves into the transparent waste bag. Close the transparent waste bag with a cable tie and put the closed transparent waste bag into the white waste bag with imprint “Zytostatika-Sondermüll” (Cytotoxic-Waste). Put the overshoes, the spectacles, the blue gloves, the respirator mask and the gown into the white waste bag. Close this bag with the second bag tie.

4 What to do then:

As a precaution, you should take a shower. If there has been skin and/or eye contact with cytotoxic drugs, consult a doctor. If applicable issue a causality report.

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5 Usage and disposal: The emergency cleaning kit must only be used once to dispose of spill cytostatic drugs. Do not reuse individual utensils either. After use, all SpillKit components should be treated as waste requiring special supervision (waste code: 18 01 08* in accordance with 2000/532/EC). Use the waste disposal utensils provided. Only approved specialist waste disposal companies may collect, transport and dispose of used kits.
6 Storage and transport: Dark (away from direct UV light and sunlight); cool (+5 to +40°C); dry; no contact with pointed and/or sharp objects. Away from equipment that can produce ozone; avoid direct contact with metals, e.g. copper, manganese, magnesium and iron; avoid contact with oil-based antiseptic phenols and their derivatives, fats, petrolatum, petroleum, paraffin or other similar compounds. Do not use oxidising cleaning agents.
7 Durability: 3 years from the date when manufactured.

8 CE marking for the PPE contained in the SpillKit: The personal protective equipment (PPE) contained in the SpillKit is certified in accordance with the PPE Regulation (EU) 2016/425 of Category II-III. EC type examination + control measures by the notified body „0426“, „2797“, „0196“, „0598“. You can find the declaration of conformity and information on PPE on our website: www.berner-safety.de. A general declaration of conformity for the SpillKit cannot be issued due to the composition of the SpillKit (PPE and accessories without CE marking).

9 Quality assurance: Tested and certified in accordance with DIN EN ISO 9001:2015 by TÜV Management Service GmbH.
10 Copyright and industrial property rights: Berner International GmbH assumes no warranty or liability for SpillKit components that are employed beyond the scope of their intended use. For translations into foreign languages, the German version of the user information is binding. The content can be changed at any time and without notice. This user information is the property of Berner International GmbH and may not be forwarded to third parties, copied, photocopied or used for promotional purposes, either wholly or in part, without written consent. Subject to technical and production-related changes. © Copyright Berner International GmbH

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9 Assurance Qualité: Testé et certifié: TÜV Management Service GmbH: DIN EN ISO 9001:2015.