BERNER Pandemie Schutz Set / Pandemic Kit Typ 1

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Siehe Punkt 3.3

1. User Information

Read the user information carefully and keep it in the **BERNER Pandemic Kit**. All personnel should be taught how to use this pandemic kit regularly. Ignoring the user information can impair the performance of the kit components and be harmful to the health of the user and the environment.

2. General information

2.1 Manufacturer

Berner International GmbH

2.2 CE-Marking

The PSA components of this set are tested acc. PSA Regulation (EU) 2016/425 for complex Category III PPE based on EN 14605 and EN 374 1-5. In addition, the protective clothing components were subjected to a test in accordance with EN 14126.

2.3 Quality assurance

Tested and certified in accordance with EN ISO 9001:2015 by the TUV Product Service GmbH.

2.4 Note

The **BERNER** Pandemic Protection Kit meets the requirements of the German Robert-Koch-Institute (RKI) for personal protective equipment in biological hazards of type 1. Further information can be found on the website of the Robert-Koch-Institute (www.rki.de/en) under the subject of infections protection / personal protection equipment.

2.5 Copyright and Industrial Property Rights

Berner International GmbH assumes no warranty or liability for **BERNER** Pandemic Kit components that are employed beyond the scope of their intended use. For translations into foreign languages, the German version of the user information is binding. The content can be changed at any time and without notice. This user information is the property of Berner International GmbH and may not be forwarded to third parties, copied, photocopied or used for promotional purposes, either wholly or in part, without written consent. Subject to technical and production-related changes. © **Copyright Berner International GmbH**

3. Description

The **BERNER** Pandemic Kit is an emergency kit for emergency personnel, who are present for a limited amount of time within a hazardous area. The **BERNER** Pandemic Kit contains a complete set of personal protective equipment, including protective overall, protective boots, gloves, safety goggles respiratory mask.

3.1 Contents **BERNER** Pandemic Kit

Order No.	Item Description	Size	Quantity
4178	BERNER Pandemic Protection Kit	M	1
4180	BERNER Pandemic Protection Kit	L	1
4182	BERNER Pandemic Protection Kit	XXL	1

Item Description	Piece / Pair
Personal Protective Equipment (PPE)	
Protective overall Tychem® C 2000	1
Protective overboots type PB-3 / PB-3-B	1
Protective gloves BERNER Manu L	1
Protective undergloves BERNER Manu N	1
FFP3 respiratory mask	1
Gas-tight safety goggles	1
Tools	
Duct tape for taping	1
Blue waste bag	1
Cable ties (for sealing the waste bag)	1

3.2 Usage

The infection protection set must only be used once. Do not re-use utensils either!

3.3 Disposal

After use, all **BERNER** Pandemic Kit components should be treated as waste requiring special supervision (waste code: 18 01 02* in accordance with 2000/532/EC). Use the waste disposal utensils provided. Only approved specialist waste disposal companies may collect, transport and dispose of used kits.

3.4 Storage and transport

Dark (away from direct UV light and sunlight); cool (+5 to +40°C); dry; no contact with pointed and/or sharp objects.

3.5 Date of manufacture

1st and 2nd digit = month, 3rd - 6th digit = year.

3.6 Shelf life

Complete **BERNER** Pandemic Kit: 3 years from date of manufacture.

3.7 LOT-Number

LOT-No.: 4/6 Digits.

4. Instructions for use

Please note and follow the correct procedure when putting the personal protective equipment:

Step 1: Put on protective coverall, close zip and stick on flap.

Step 2: Put on protective overboots, pull over protective coverall and tie strap.

Step 3: Take respiratory mask, by hold the mask in your hand with the straps hanging down (1). Guide the first strap over your head as far as the nape of your neck. Place the mask under your chin. Guide the second strap over the back of your head (2). Adjust the nosepiece to fit your nose (3). Cover the half-mask with both hands and breathe out strongly (4). The mask material will move up and down. If air escapes at the edge of the mask, move the mask into the correct position and adjust the nosepiece to fit your nose. Please note that facial hair and scars can impair the fit of the mask.

Step 4: Put on the safety goggles. Use the head strap to adjust them and ensure a secure fit. Safety goggles can be worn over corrective glasses. Please check the closeness before start working.

Step 5: First don the light blue **BERNER** Manu N protective gloves and pull the cuffs over the coverall sleeve end. Then don the blue **BERNER** Manu L protective gloves over the under gloves.

Step 6: Using the duct tape tape the **BERNER** Manu L protective gloves to the overall. Overlap the tape pieces.

Step 7: Now use the tape to join the safety boots to the coverall.

Step 8: Tape the respiratory mask and safety goggles to the hood of the overall. Use short tape pieces and ensure that no creases appear in the fabric or tape since this could act as channels for contamination.

5. Product Properties

5.1. Protective coverall Tychem® C and protective overboots type 3/3-B / PB-3B

Properties

Maximum protection and wearing comfort: Type-tested and certified as complex PPE category III; chemical protective equipment tested to EN 14605, EN 1073-2. Additional tested as protective clothing against biological hazards acc. to EN 14126. Optimum personal - and product protection; impermeable to liquids, antistatically treated, comfortable to wear. **Protective coverall:** High neckline, zipper, taped seams, elasticated sleeves, legs and waist, hood. **Protective overboots:** Elasticated leg with tie strap. **Application range:** Protective overboots and coverall are suitable for protection against hazardous substances. Keep away from naked flames and heat sources. **Before use:** Check for any damage! Do not use damaged coverall or overboots!

See the enclosed leaflet for detailed information and further instructions on the use of the coverall!

Versions protective overall

Pandemic Kit	Size	Chest (cm)	Height (cm)	Colour
4178	M	92 - 100	168 -176	yellow
4180	L	100 – 108	174 – 182	yellow
4182	XXL	116 – 124	186 – 194	yellow

Versions protective overboots

Size	Uni	Colour	yellow
------	-----	--------	--------

5.2. Protective gloves BERNER Manu L and Manu N

Properties

Maximum protection and comfort: type tested and certified as complex highest level PPE of the category III; good grip, good tactile sensitivity. **Area of application:** Protective gloves for handling chemicals and biological agents (e.g. bacteria and viruses). **Protective capacity:** No guarantee for all chemicals! In case of exposure to biological hazards, which do not correspond to the degree of imperviousness of the protective clothing, bio-contamination of the wearer is possible. **Before use:** Check for any damage! Do not use damaged gloves!

Types

Pandemic Kit	4178	4180	4182	4178	4180	4182
Size	8 / M	8.5 / L	9 / XL	8 / M	8.5 / L	9 / XL
Protective glove	Manu L			Manu N		

Material properties

5.2.1 Protective glove BERNER Manu L

Natural latex (latex allergen < 50 µg/g); low protein (protein = 17 µg/g); powder free in accordance with TRGS 540; latex and carbamates can trigger allergies.

Material thickness

Measurement points	Material thickness d (measured twice)
Finger, 15 mm from the end of the tip	≥ 0,96 mm
Middle of the palm	≥ 0,86 mm
Shaft, 25 mm from the end of the shaft	≥ 0,48 mm

Protection against chemical dangers; permeation in accordance to EN 374-1:2016, Test method EN 16523-1:2015;

Degradation in accordance to EN 374-4:2013.

For the following chemicals breakthrough times¹⁾ [min]/performance classes²⁾ (1-6) and degradation percentages³⁾ were determined:

Chemical	Breakthrough time ¹⁾ [min]	Performance class ²⁾ (1-6)	Degradation ³⁾
40% Sodium hydroxide (K)	> 480	6	-6.5%
37% Formaldehyde (T)	> 480	6	-3.1%
30% Hydrogen peroxide (P)	> 480	6	-12.5%

1): Movement of a chemical through a material at the molecular level.

2): At a permeation rate of 1µg/min cm²

3): The power class does not reflect the actual duration of protection at the workplace!

Protection against penetration in accordance to EN 374-2:2014

Tested according to EN 374-2:2014. Test conditions fulfilled. Air leak test only.

Protection against biological hazards: Resistance to viruses, bacteria & fungi

Tested according to EN ISO 374-5:2016. According to the current state of knowledge, it can be assumed that effective protection against microbiological hazards exists with adherence to penetration and an AQL=0.65.

CE-Marking

In accordance with the According to the PSA regulation (EU) 2016/425 for complex PSA of category III, based on EN 374 part 1-5, EN 420. Certification and control measures by the notified body „0086“. Declaration of conformity available at: www.berner-safety.de.

5.2.2 Protective glove **BERNER Manu N**

Latex-free protective glove made from nitrile; powder-free in accordance with TRGS 540

Material thickness

Measuring points	Material thickness d (measured double)
Finger, 15 mm from the end of the tip	≥ 0,27 mm
Middle of the palm	≥ 0,21 mm
Shaft, 25 mm from the end of the shaft	≥ 0,18 mm

Protection against chemical hazards; permeation according to EN 374-1:2016, Test method EN 16523-1:2015; Degradation according to EN 374-4:2013.

Breakthrough times¹⁾ [min]/performance classes²⁾ (1-6) and degradation³⁾ were determined for the following chemicals:

Chemical	Breakthrough time ¹⁾ [min]	Performance class ²⁾ (1-6)	Degradation ³⁾
40% Sodium hydroxide (K)	> 480	6	-34.0%
37% Formaldehyde (T)	> 240	5	-8.0%
30% Hydrogen peroxide (P)	> 120	4	-0.8%

Protection against penetration according to EN 374-2:2014

Tested according to EN 374-2:2014. Test conditions fulfilled.

Protection against biological hazards: Resistance to viruses, bacteria & fungi

Tested according to EN ISO 374-5:2016. According to the current state of knowledge, it can be assumed that effective protection against microbiological hazards exists with compliance with penetration and AQL=1.5.

CE-Marking

In accordance with EU PPE regulation (EU) 2016/425 for complex PPE (category III), based on EN 374, parts 1-5; EN 420. EC type test, certification and control measures by Notified Body "2777". Declaration of conformity available at: www.berner-safety.de

5.3 FFP3 Respiratory mask

Properties

Half-mask with high filtering capacity, tight-fitting and leakproof. Its soft material, special shape and nose clip ensure it can be easily adjusted to fit the face. Minimal respiratory resistance and the exhalation valve make it extremely comfortable to wear. The threaded elastic strap makes it easy to put on and take off. **Area of application:** protection from solid and liquid particles, e.g. for handling CMR drugs. **Area of application:** Protection from solid and liquid particles, e.g. for handling CMR drugs. **Before use:** Check for damage. Do not use damaged masks. **Limitation:** laboratory test results can differ from values obtained in practice considerably and, as a result, the useful life of the mask may increase or decrease. Facial hair and scars can impair the fit of the mask and therefore the level of protection. (If nec. use vaseline).

Type

Size	Universal
Colour	silver grey

Material properties

Latex-free	complete mask
Dimension	125 mm x 135 mm
Weight excl. packaging	18 g

Degree of particle filtration

Degree of particle filtration	Minimum degree of filtration
Sodium chloride	> 99%
Paraffin oil	> 99%

Dolomite dust test for FFP3 - respiratory masks (Dolomite DRB4/15b testaerosol)	passed
Gases	not suitable

CE-Marking

In accordance with EU PPE regulation (EU) 2016/425 for complex PPE category III, based on EN 149. Type-testing, certification and control measures by the notified body "0194".

Notified body 0194

INSPEC International Ltd., 56 Leslie Hough Way, Salford, Gt. Manchester, M6 6AJ, England

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5.4 3M Fahrenheit™ safety goggles, gas-tight

Properties

Full vision goggles; non-fogging and can be worn over corrective glasses. The elastic strap ensures a firm and fit size. **Area of application:** Protection from mechanical risks, liquids and coarse dust. **Protection capacity:** Does not provide protection from high-speed particles, laser beams, temperatures of over 55°C, arcing faults, molten metals. **Before use:** Check for any damage! Do not use goggles if label is missing or illegible.

Version

Size	Universal
Colour	Crystal blue

Marking of the frame and lens

AOS EN 166 F 3,4,9 CE 0194	Protection against mechanical risks, liquids, large particles
2C -1,2 AOS 1 F 9 N EN 166 CE 0194	Non-fogging, colourless plastic lens

CE-Marking

In accordance with the EU PPE regulation (EU) 2016/425 for complex PPE category III based on EN 166. Type-tested and certified by the notified body „0194“.

Notified body 0194

INSPEC International Ltd, 56 Leslie Hough Way, Salford, Greater Manchester, M6 6AJ, Great Britain.

Manufacturer

3M Cente, Bracknell, Berkshire, RG12 8HT, Great Britain.

For further details please see the enclosed leaflet!

5.5 Further components of the BERNER Pandemic Kit

Duct tape	Colour silver, for taping the PPE, removable
Waste bag	Colour blue, material polyethylene
Cable ties	For sealing the waste bag

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Put on protective coverall and fix the self-adhesive zip-cap

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Put on rubber boots and dress the coverall over it

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Don the thin, light blue Manu N protective gloves

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Now don the dark blue Manu L protective gloves

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Seal the connection between gloves and coverall with Duct tape

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Seal also the Connection between coverall to boots

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Last seal the remaining gap between mask/goggles + hood with Duct tape